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EN - INSTRUCTIONS FOR USE  
 Single-use filtering face-piece respirators

#### IMPORTANT:

Single-use respirators are certified in accordance with the European PPE Regulations 2016/425 on safety requirements for personal protective equipment and classified to one of three categories: FFP1 FFP2 or FFP3. Before reading the following information, check the category to which the respirator belongs - this is indicated on the packaging and on the respirator. If the mask is marked D, it has passed the dolomite clogging test and suitable for use in very dusty environment. It is the user's responsibility to ensure that the respirator provides the necessary level of protection for the type and concentration of the contaminant(s) in the area where the respirator is intended for use.

#### DIRECTIONS FOR USE:

- Fit the respirator and check - leak-tightness BEFORE entering the contaminated area.
- Wear the respirator for the whole duration of exposure to contaminants.
- Use the respirator in accordance with applicable health and safety regulations.

Discard the respirator and replace with a new one if:

- the respirator is removed while in a contaminated area
- excessive clogging of the respirator causes breathing difficulty or discomfort
- the respirator becomes damaged
- (for respirators protecting against vapours) - the smell of vapours present becomes detectable.
- Leave the contaminated area if dizziness, irritation or other distress occurs.
- Only for single use, No maintenance necessary. Do not store and/or re-use after single use.
- Discard the respirator after single use (one shift, maximum).
- Keep un-used respirators in their closed box and store in a dry non-contaminated area between -20°C and +40°C at relative humidity under 75%.

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Hair and ears should be covered with a suitable cap to minimize the spread of hair and other particles. If there is a risk of liquid splashes, use a 4-layer splash-resistant mask with a visor or a 4-layer splash-resistant mask and protective glasses / visor

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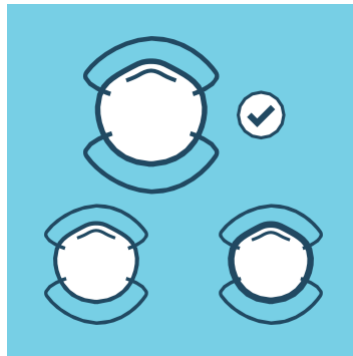
Wash your hands before wearing a mask

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Choose a mask size and model that fits you well

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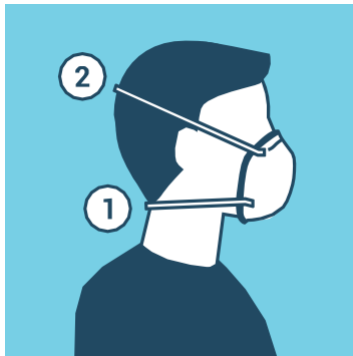
Hold the mask in your hand and place it on nose, mouth and chin

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Position the bottom strap under your ears and the top strap at the back of your head

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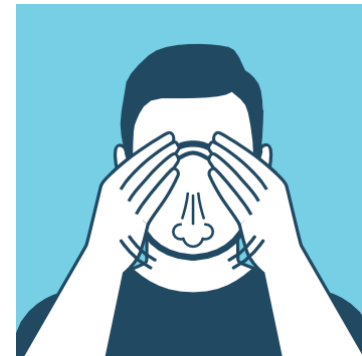
Press the metallic wire to fit the shape of your nose and face

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Perform a fit check: Place hands over the respirator and exhale, adjust if there are leaks

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